

## Proloquo2go Backup to Dropbox



1. Install the Dropbox app on your iPad.
2. Open Proloquo2go and Tap Settings icon.
3. Tap **Dropbox** tab.




4. Turn on **Dropbox Integration**




5. Tap **Open** and **Allow** when prompted.

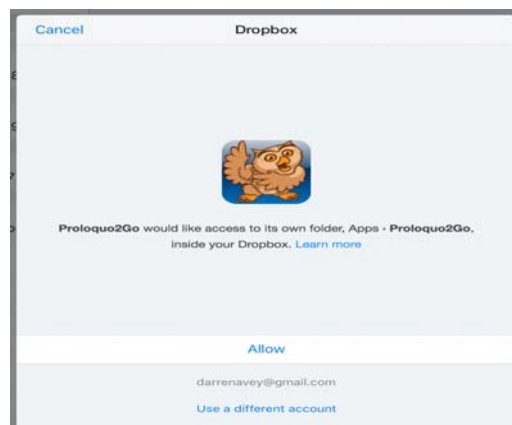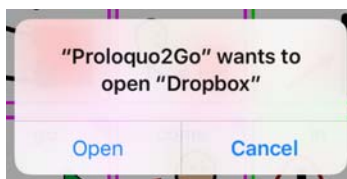



6. Tap **Backup** tab in Proloquo Settings and Backup the user you wish to move to another iPad.

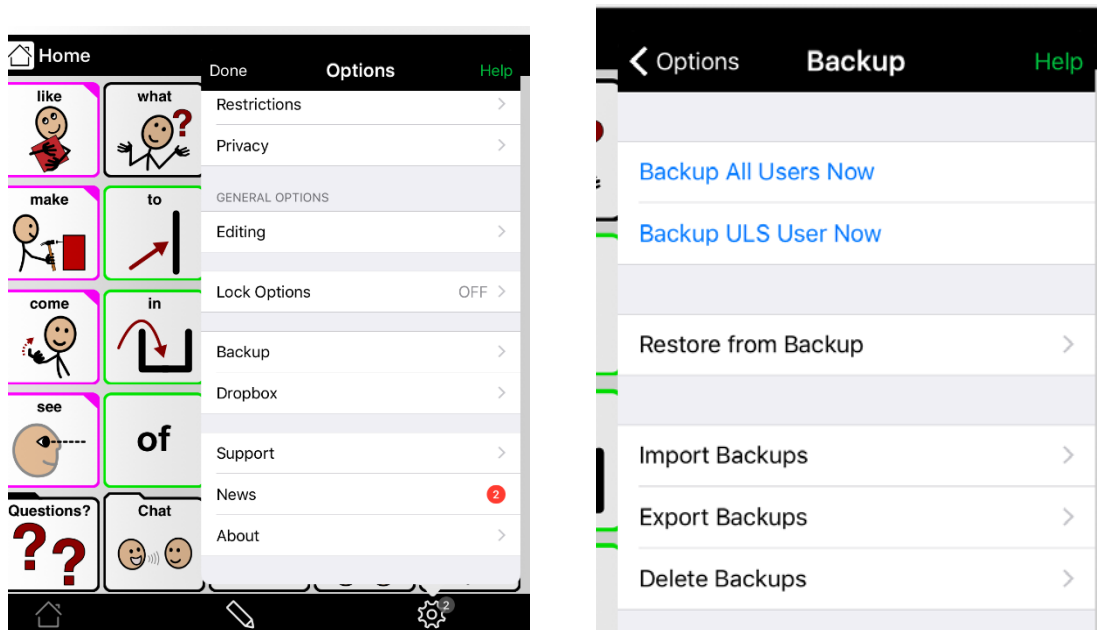

7. Tap **Export Backups**  
8. Tap the Backup file you wish to export

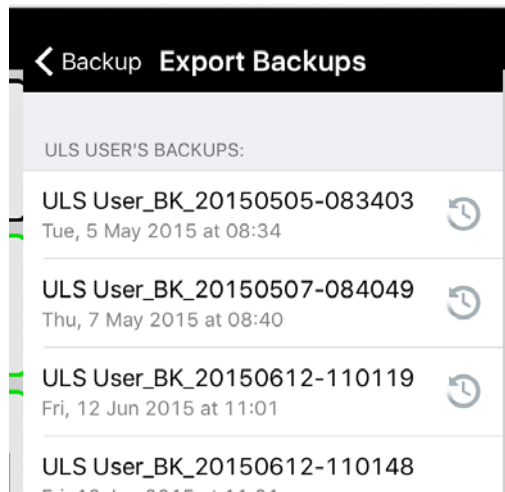


9. Tap **Export to Dropbox**

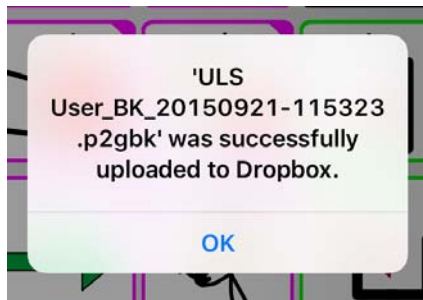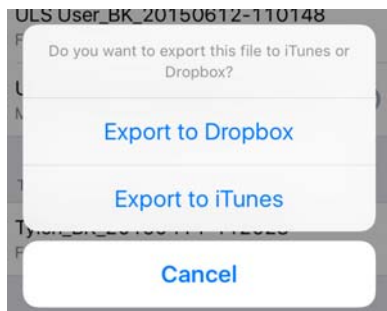


## Opening a Proloquo File from Dropbox on an iPad



1. In the Dropbox app tap the **Files** section at the bottom of the app
2. Tap the **Apps** Folder
3. Tap the **Proloquo2Go** Folder

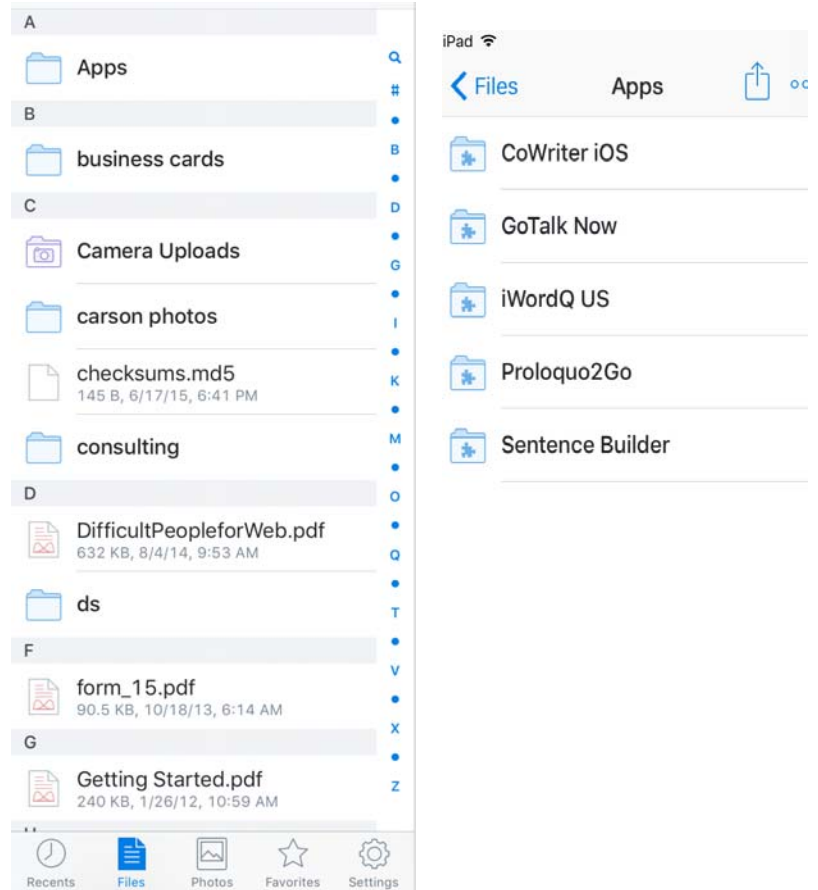


4. Tap the file name you wish to open on the new iPad
5. Tap the arrow and the top of the page

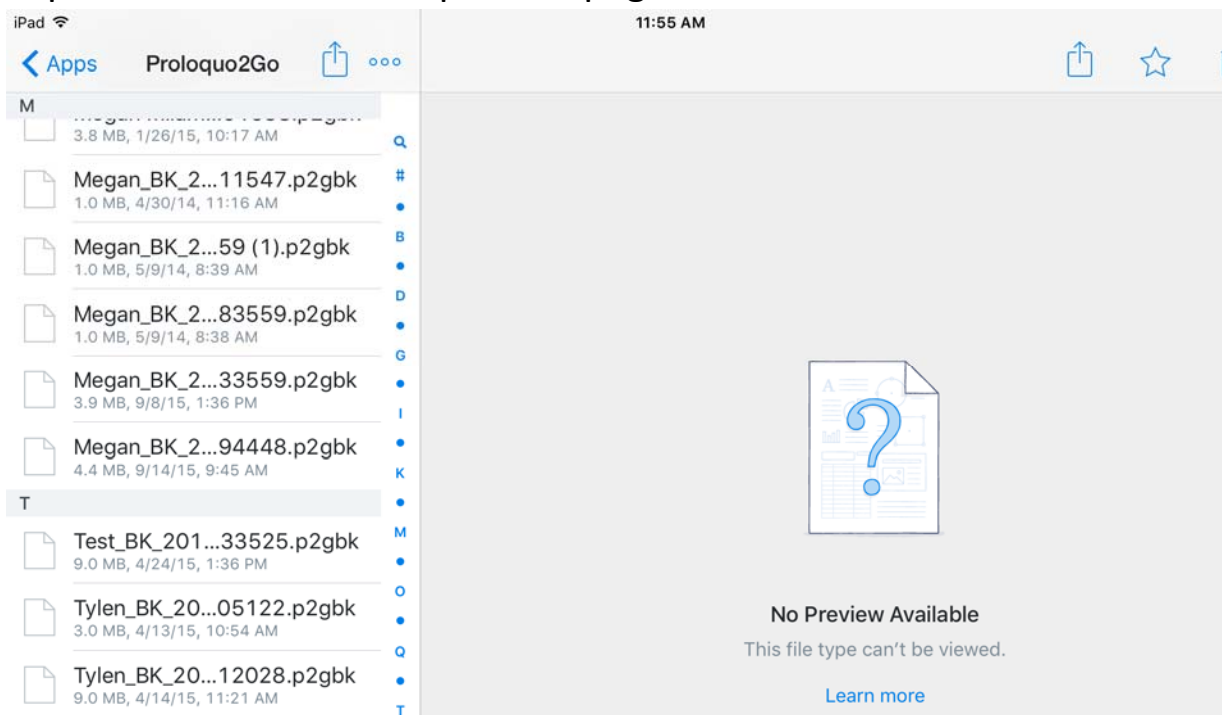

6. Tap **Open In..** and then Tap Proloquo2Go icon.

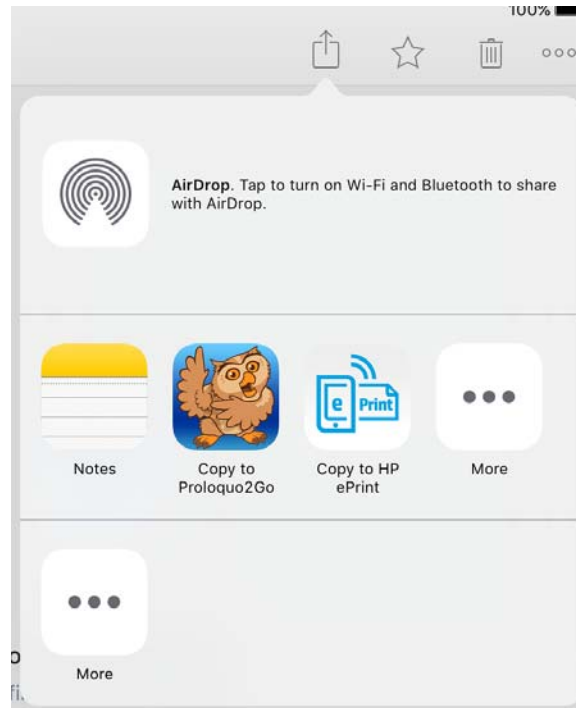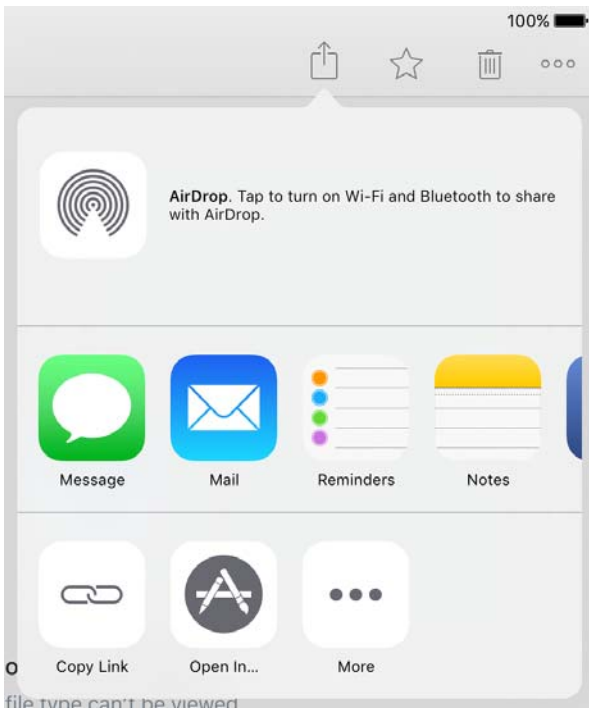

7. When Proloquo2Go opens you will press **Replace**.

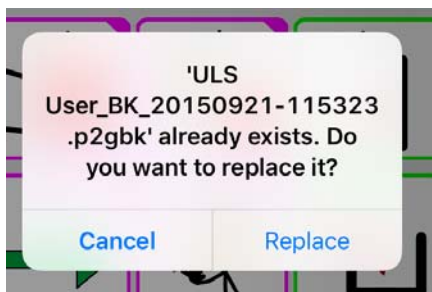

This will replace the current user that is open in Proloquo2Go. Be careful if you have more than one user on your iPad.